

KFN Explorer 4



MANUAL

Manufacture by:
KFN ApS
Bondesvadvej 15
DK 8300
Odder
Denmark
+4586142265

www.kfn-aps.dk kfn@kfn-aps.dk

KFN ApS - Bondesvadvej - 15 DK - 8300 - Odder - Denmark
+4586142265

www.kfn-aps.dk kfn@kfn-aps.dk



CONTENTS

General Information

1. Introduction
2. Recommendations
3. Declaration of conformity
4. Warranty
5. Service and repair

Safety measures

1. General safety

Assembly instructions

1. Stroller
2. Footrest adjustment

Accessories

1. Hood
2. Storm cover
3. Seat insert

Specifications

General information

1. Introduction

Thank you for buying a KFN Explorer 4.

This quality product has been developed to enhance everyday lifestyles for those families with Special Needs or Mobility requirements.

The manual is designed to help you with the safe use and care of your KFN Explorer 4. Please read it carefully to ensure you use the product as recommended.

If you require any further information and advice, please contact your dealer.

Enjoy your stroller!

2. Recommendations

The KFN Explorer 4 is designed for passengers with special needs with the recommended age range 4 to 10 years for the junior and 10 years to small adult for the senior.

The purpose of the stroller is for the family to continue their outdoor activities of walking, jogging, trekking and off road hiking. The four wheels give maximum stability on a variety of terrains.



2. Recommendations

The KFN Explorer 4 is designed for passengers with special needs with the recommended age range 4 to 10 years for the junior and 10 years to small adult for the senior.

The purpose of the stroller is for the family to continue their outdoor activities of walking, jogging, trekking and off road hiking. The four wheels give maximum stability on a variety of terrains.



The KFN Explorer 4 is not suitable for all day use, it is intended as a leisure product to transport passengers over rough terrain.

3. Declaration of conformity

This stroller conforms with the guideline tests as set out in the EC Medical Directive 93/42/EEC and therefore is CE marked.

4. Warranty

The guarantee covers the product for 12 months when used in accordance with the manufacturer's recommendations. Any misuse will result in the guarantee becoming void.

5. Service and repair

During the warranty period, service and repair should be carried out by an authorised dealer using KFN parts. In case of doubt, please contact your supplier.

Safety measures

1. General Safety

Please read the manual carefully as careless use may cause injury or death.

The safety of the passenger is your responsibility and therefore please ensure the harness is securely fastened before use.

Regularly check moving parts and ensure fastenings are all fully tightened.

Check brakes and brake pads are in good condition and fully functioning.

Engage brakes when loading and unloading passenger.

Do not hang heavy items on the handle bar as this will affect the point of balance of the stroller and may cause instability.



Ensure wheels are correctly assembled and engaged.



Do not remove any wheels when the stroller is occupied.

To maintain the stability of the stroller, the passenger must not stand, sit on knees or ride facing against the direction of travel.



Metal components if left in direct sunlight, may become hot to the touch.

Extra care needs to be taken when using the stroller on an incline. Always park with the rear wheels at the highest point with the brakes engaged! This is to ensure the passenger does not fall out, the stroller tip backwards or roll away. Likewise when walking across a severe incline, be aware the stroller could tip sideways.

If lifting the Explorer 4 when loaded, carry by holding the frame at the following points:

- Both sides of the frame near the front wheels
- The rear axle
- The handlebar

Tyre pressures

Properly inflated tyres will increase the handling performance and manoeuvrability of the stroller - the pressure must be 29 Psi (or 2 Bar) per tyre.

Never leave the passenger unattended in the stroller – even with the harness secured and the brakes engaged.

The maximum capacity of the storage pocket is 3 kgs.

The stroller is intended for single occupant use only.

The maximum load capacities for the strollers are as follows:

Junior	60 kgs
Senior	80 kgs



WARNING – KEEP PACKAGING OUT OF THE REACH OF CHILDREN

KFN ApS - Bondesvadvej - 15 DK - 8300 - Odder - Denmark

+4586142265

www.kfn-aps.dk

kfn@kfn-aps.dk



ASSEMBLY INSTRUCTIONS

1. Stroller

Your stroller will arrive in flat pack form consisting of:

- 1 x chassis including sun hood
- 2 x rear 20" wheels
- 2 x front 16" wheels
- 4 x quick release wheel axles

Frame assembly

Locate the two black levers on either side of the frame.

Unscrew the bolts using the levers.

Place the ends of the support arm in line with the revealed hole and replace the bolt using the lever to tighten (the lever can be placed in any position once the bolt is tightened).

Repeat for the other support arm.

Wheel assembly

For all wheels - place wheel axle through centre of wheel by pressing the sprung end.

Before attaching the wheels to the stroller, ensure the brakes are not engaged.

Rear wheels – depress the black button and push the wheel axle through the tube. Check the cut out wheel hub is on the inside next to the chassis. Ensure the wheel is attached by pulling on the wheel

Front wheels – depress the black button and push the wheel axle through the tube on the outside of the frame, ensure the wheel is attached by pulling on the wheel.

Brake operation

When parking the stroller, squeeze the brake handle and use the red lever to lock the brake on. To deactivate the lock, squeeze the handle to release the red lever.

When using the stroller on a steep incline, the brakes can be used to control the stroller descent and also assist with right and left turning by using the relevant right or left brake lever.





2. Footrest adjustment

Using an allen key, loosen the bolts either side of the footrest and slide to the correct position. Tighten the bolts ensuring the footrest is level.



ACCESSORIES

1. Hood

When the sun canopy is not in use, it can be moved out of the way by hanging behind the handlebar at the back of the seat.

To keep the sun canopy in place when required, use the Velcro strap to secure around the centre of the handle bar and attach the Velcro at the sides.



2. Storm cover

The hood when used with the storm cover will keep the passenger protected from wind and rain.

To attach the storm cover, use the Velcro to attach down both sides of the stroller.




3. Seat insert

The four compartment Seat Insert is attached by inserting the safety harness through the seat and then fastening to the stroller using the ties at the top corner.



To adjust the seating position and support of the passenger, the cushion filling can be removed as required through the zips in each of the compartments.

The cover of the Seat Insert is cotton and can be removed for washing.

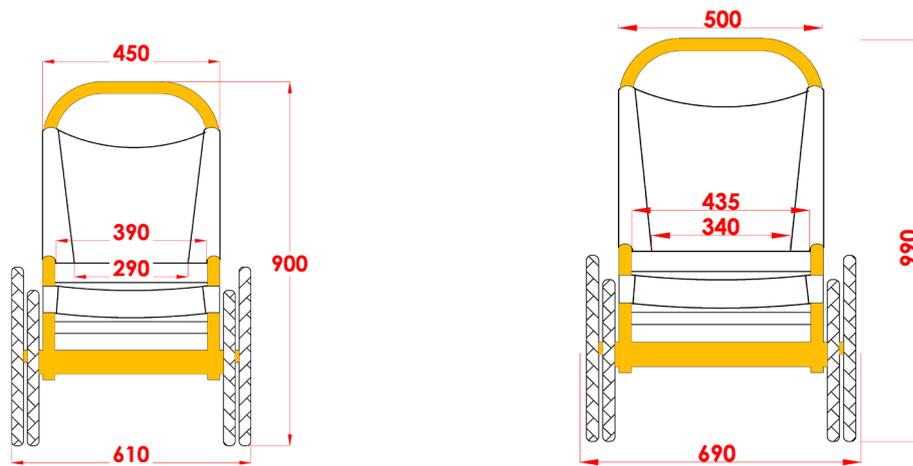

MEASUREMENTS mm	JUNIOR	INSERT	SENIOR	INSERT
Seat depth	300	230	450	370
Seat width	390/290	290	435/340	330
Leg rest length	190/300		200/480	
Back height	630	550	680	600
Back rest angle	82 deg		82 deg	
Stroller weight	12.5 kgs	13.5 kgs	13 kgs	14 kgs
Stroller capacity (maximum load)	60 kgs		80 kgs	
Maximum width (with wheels)	610		690	
Handle height	900		990	
Folded length	1250		1500	

